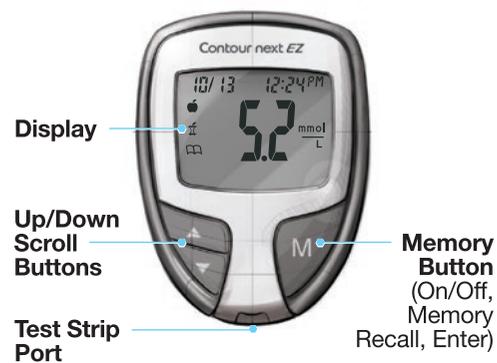
Contour®
next EZ
Blood Glucose
Monitoring System

NO CODING

QUICK REFERENCE GUIDE

Your CONTOUR®NEXT EZ Meter

Use only the CONTOUR®NEXT Test Strip

Gray End:
Insert this end into the test strip port.

Sample Tip:
Blood sample pulled in here.

Memory Button
(On/Off, Memory Recall, Enter)

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
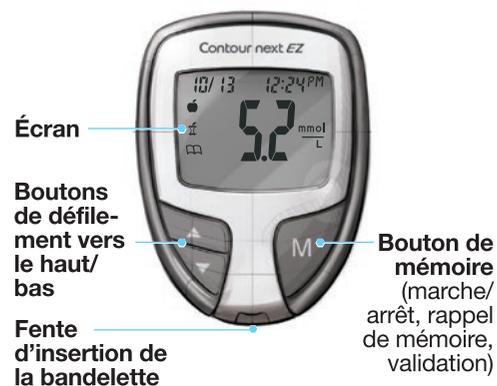
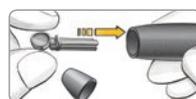
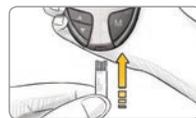
Basic Testing

Getting Ready to Test

1. Wash and dry your hands well before testing.
2. Insert the gray end of the test strip into test strip port on the meter.
3. The meter will turn on and a test strip with a flashing blood drop will appear on your meter display letting you know the meter is ready to test.
4. Remove the endcap from the lancing device.
5. Loosen the round protective cap on a lancet by rotating it ¼ turn, but do not remove it.
6. Insert the lancet firmly until it comes to a complete stop.

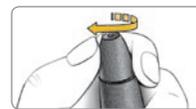7. Twist off the round protective lancet cap.
8. Replace the endcap.
9. Adjust the endcap dial to your preferred puncture depth setting.

CAUTION: The lancing device is intended for use by a single patient.

Getting the Blood Drop

1. Press the lancing device firmly against the puncture site and press the release button.
2. Stroke your hand and finger towards the puncture site to form a drop of blood.

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Testing Your Blood

1. Use only CONTOUR®NEXT test strips with the CONTOUR®NEXT EZ meter.
2. Touch the tip of the test strip to the drop of blood. The blood is pulled into the test strip.
3. Hold the strip in the blood drop until the meter beeps.
4. Your result is displayed* after a 5 second countdown.
5. To turn your meter off, remove the test strip. Dispose of the test strip as medical waste.

*Your meter is preset and locked to display results in mmol/L. If your result is in mg/dL, contact Customer Service at 1-800-268-7200.

Fingertip Testing

See your CONTOUR®NEXT EZ User Guide and lancing device insert for complete instructions.

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- CAUTION:**
- The lancing device provided with your kit is not suitable for use by health care professionals. It must not be used on more than one person due to risk of infection.
 - To remove the lancet from the lancing device refer to the detailed instructions in the lancing device insert.
 - The lancing device graphics are for illustration purposes only.

For more information, go to www.diabetes.ascensia.com or contact Customer Service at: 1-800-268-7200.

ASCENSIA
Diabetes Care

Manufactured for:
Ascensia Diabetes Care Holdings AG
Peter Merian-Strasse 90
4052 Basel, Switzerland

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Potential Biohazard

All blood glucose measuring systems are considered biohazardous.

All products or objects which come in contact with human blood, even after cleaning, should be handled as if capable of transmitting viral diseases.

The user should follow the recommendations for prevention of blood-borne transmissible diseases in health care settings as recommended for potentially infectious human specimens.

See your User Guide for complete instructions.

© 2016 Ascensia Diabetes Care Holdings AG. All rights reserved.

Ascensia, the Ascensia Diabetes Care logo, Clinilog, Contour, Glucofacts, and the No Coding logo are trademarks and/or registered trademarks of Ascensia Diabetes Care Holdings AG.

TM see www.ascensiad diabetes.ca/en/tm-mc

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Using Advanced Mode

Setting Your Meter to Advanced Mode

To set your meter to Advanced Mode, and take advantage of meal markers and other available features:

1. Press **M** to turn your meter on.
2. Press and hold **▲** or **▼** for 3 seconds.
3. L-1 will flash.
4. Press **▲** or **▼** to change to L-2.
5. Press **M** to set.

Find out how these additional features can help you manage your diabetes.

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After you perform a test and see your results, use the markers on the display:

To Mark Results	You Will See	To Set	You Will See	What It Means	Using The Reminder
Pre-Meal Marks a test result that was taken before a meal. Press ▲ or ▼	(Flashing)	Press M		Result is marked as a pre-meal test. Press M again to set a reminder to run a post-meal test. →	 When the reminder sounds, the meter turns on, and you will see your pre-meal test result.
Post-Meal Marks a test result that was taken after a meal. Press ▲ or ▼	(Flashing)	Press M		Result is marked as a post-meal test.	
Logbook Marks any result as being unique or different in some way. Press ▲ or ▼	(Flashing)	Press M		Result is marked in memory as unique. You should note this in your logbook.	

Sets a reminder (pre-set at 2 hours) to run a post-meal test. See User Guide for complete instructions on how to use the markers and adjust the reminder times.

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Error Codes and Symbols

See your User Guide for a complete list of Error Codes.

E1	Temperature out of range.
E2	There was a problem completely filling the test strip. Retest using a new test strip.
E3	The meter is sensing a used test strip or the wrong control solution was used. Retest using a new test strip.
E11	Abnormal result. Retest using a new test strip.

Ascensia, the Ascensia Diabetes Care logo, Clinilog, Contour, Glucofacts, and the No Coding logo are trademarks and/or registered trademarks of Ascensia Diabetes Care Holdings AG. TM see www.ascensiad diabetes.ca/en/tm-mc

For information on patents, and related licenses, see: www.patents.ascensia.com

© 2016 Ascensia Diabetes Care Holdings AG. 85475711
All rights reserved. Rev. 08/16

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