

Contour
Evolving with you

GlucoContro.online

An easy-to-use web portal
to help guide your patients
diabetes management*



Invite your patients to share
their blood glucose results through
our secure CONTOUR®CLOUD or
plug in directly to download results.

www.glucocontro.online

 **GlucoContro**
.online

*Patients need to be registered in GlucoContro.online and
consent to sharing their data with their healthcare professional.

Easy access to patient blood glucose information

Quick registration with an easy-to-understand dashboard



Dashboard summary statistics with complete list of **registered patients**



Fast and easy access to CONTOUR®DIABETES app data stored in CONTOUR®CLOUD.



Personalized information about each **registered patient** (blood glucose target ranges, time frames)



Generate reports & gain insights Including AGP report*, in PDF or print



John Lee ✓

61 y/o

Type 2



Quick data summary Colour-coded charts, as easy as **green, yellow, red.**



A secure environment for your patients to share information with you+



Download results from CONTOUR® meters without administrator permissions




* Ambulatory glucose profile

+ Patient should consult with their healthcare professional prior to making any changes to treatment regime.

A new way to view blood glucose results

Colour-coded dashboards help interpret and understand blood glucose results


Quick access menu:

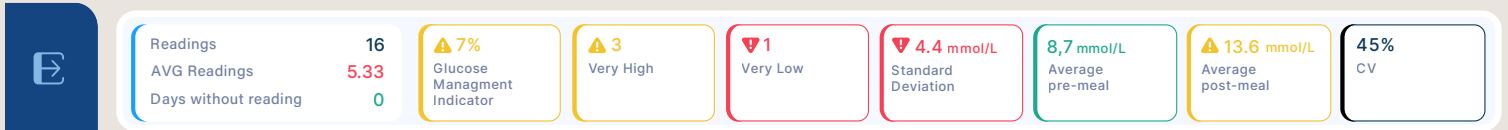

Quick Summary: Relevant information for decision making



Glucose target ranges:
View percentage of readings in each glucose target range

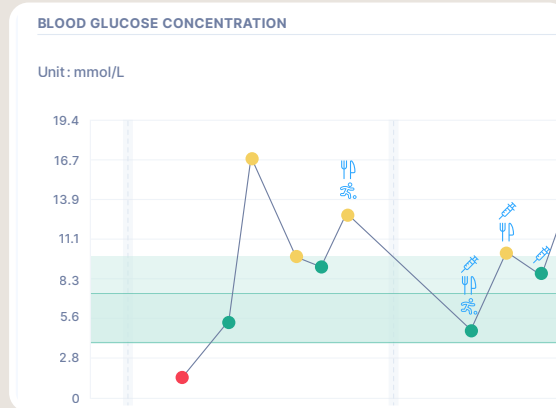

Glucose results distribution:
View variability of glucose readings



Results Summary table:
Simple display of all collected data for each day

Blood glucose results:

	Total count	Average	Median	Std. dev.	Highest	Lowest	High	Target	Low
All	16	9.5 mmol/L	9.9 mmol/L	4.3 mmol/L	16.8 mmol/L	2.3 mmol/L	43.75%	43.75%	12.5%
Fasting	3	3.8 mmol/L	3.7 mmol/L	1.6 mmol/L	5.4 mmol/L	2.3 mmol/L	0%	33.33%	66.67%
Pre-meal	3	8.7 mmol/L	9.8 mmol/L	2.4 mmol/L	10.3 mmol/L	5.9 mmol/L	33.33%	66.67%	0%
Post-meal	5	13.6 mmol/L	15.0 mmol/L	3.4 mmol/L	16.8 mmol/L	9.4 mmol/L	80%	20%	0%
No flag	5	9.3 mmol/L	9.9 mmol/L	2.9 mmol/L	12.7 mmol/L	4.6 mmol/L	40%	60%	0%

Results summary: For a more detailed summary of patient blood glucose (BG)

Simple onboarding and adding patients:

1 Register in 3 simple steps:

1. Visit glucocontro.online and click Create an Account
2. Fill out information
3. Wait for the approval from the clinic administrator (it may take up to 48 hrs)

Create an account

Professional
If you are a healthcare professional.

Staff
If you work at a clinic, but you are not a healthcare professional.

Personal
If you were diagnosed with diabetes.

Caregiver
If you are caring for a person diagnosed with diabetes.


[Continue](#)

I already have an account. [Sign in](#)

2 3 options to add a new patient:

1 – Clinic Code

Give your unique clinic code to your patients. They can use this code when they log into GlucoContro.online, to enroll in your clinic.

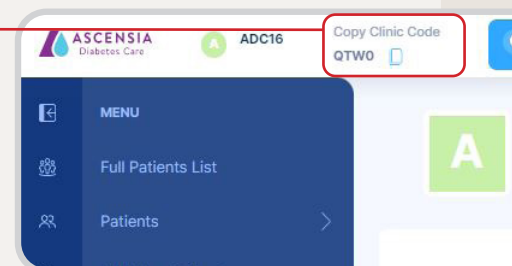

2 – Manual Entry

Enter their first and last name, country, health ID and date of birth. These are the minimum details required to add a new patient. You also have the option to send an e-mail notification to your patient. Once complete press the “Add new patient” button and they will be displayed under the patient list.

Add new patient

Manual QR Code

First name Last name

I want to send an e-mail invitation to this patient

Country

personalIdentifierType personalIdentifierValue


10 digits 10 digits

Optional information

[Add new patient](#)

3 – QR Code

Click on the QR code and ask your patient to scan it from your screen using their mobile device.

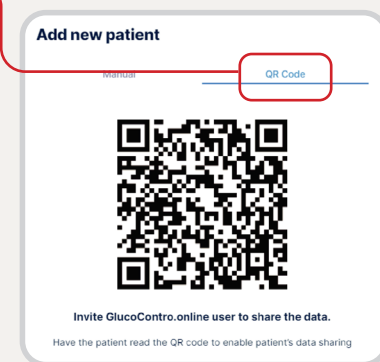

GlucoContro.online Help Center:
support@glucocontro.online

©TM see ascensiadiabetes.ca/tm-mc. © 2023 Ascensia Diabetes Care Canada Inc. All rights reserved.

Google Play and the Google Play logo are trademarks of Google LLC. Apple and the Apple logo are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc. Compatible devices can be found at <http://compatibility.contourone.com>. SdNcenter is the legal manufacturer and developer of GlucoContro.online and is partnering with Ascensia to market and deliver the tool to HCPs and Patient users.

0323-0183

